




MUBY FLOW

Hands-free Double Electric Breast Pump

—— User Manual ——

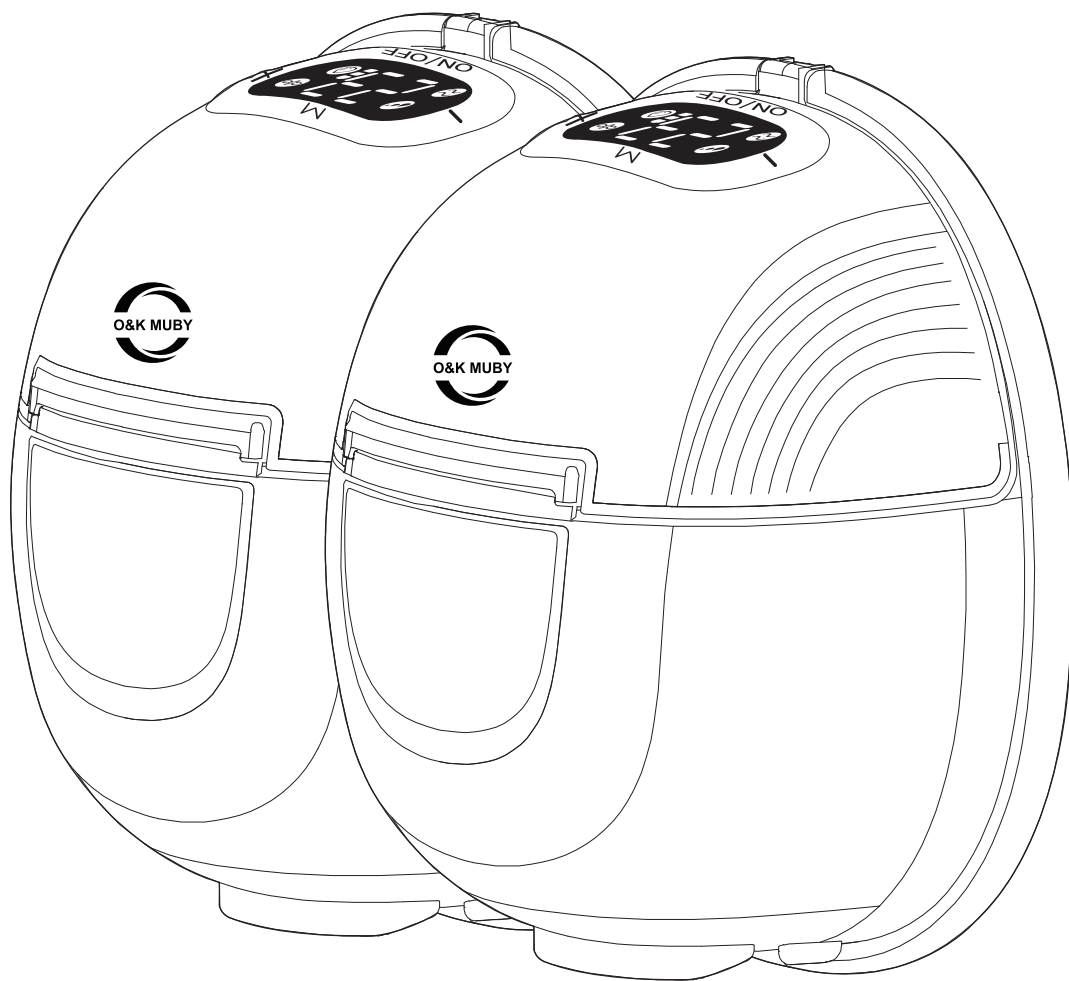

Four Mode Switch · Rechargeable Lithium Battery · Low Noise · BPA Free

FEATURES

4 MODES & 9 LEVELS: Mothers can try different modes (Multitronic Mode, Massage Mode, Express Mode & Suck simulate mode) and levels 1~9 to select the best way to pump.

LOW NOISE: The breast pump electric is made with better design and craftsmanship. Noise is only 45dB.


PORTABLE: It is light and fits in the bra, the rechargeable lithium battery allows you to pump anywhere and anytime.

COMPONENTS

Please check the following accessories before use, if there are missing or damaged pieces, please contact the supplier.

	Component Name	Materials	Operating Temperature	Sterilization Instruction
Ⓐ	Motor	/	/	Wipe with soft duster
Ⓑ	Diaphragm	silicone	-20℃ / -4℉ ~ 200℃ / 392℉	Rinse in water/Not suitable for prolonged sterilization
Ⓒ	Milk Collector	PP	-20℃ / -4℉ ~ 120℃ / 248℉	Boiling water/Microwave/Dishwasher/UV safe
Ⓓ	Duckbill Valve	silicone	-20℃ / -4℉ ~ 200℃ / 392℉	Rinse in water/Not suitable for prolonged sterilization
Ⓔ	Flange	silicone	-20℃ / -4℉ ~ 200℃ / 392℉	
Ⓕ	19/21mm Flange Insert	silicone	-20℃ / -4℉ ~ 200℃ / 392℉	
Ⓖ	Bra Adjustment Buckle	/	/	Wipe with soft duster
Ⓗ	Charging Cable	/	/	
	Instructions	/	/	/


Please clean and disinfect all parts that come into contact with food before and after each use.

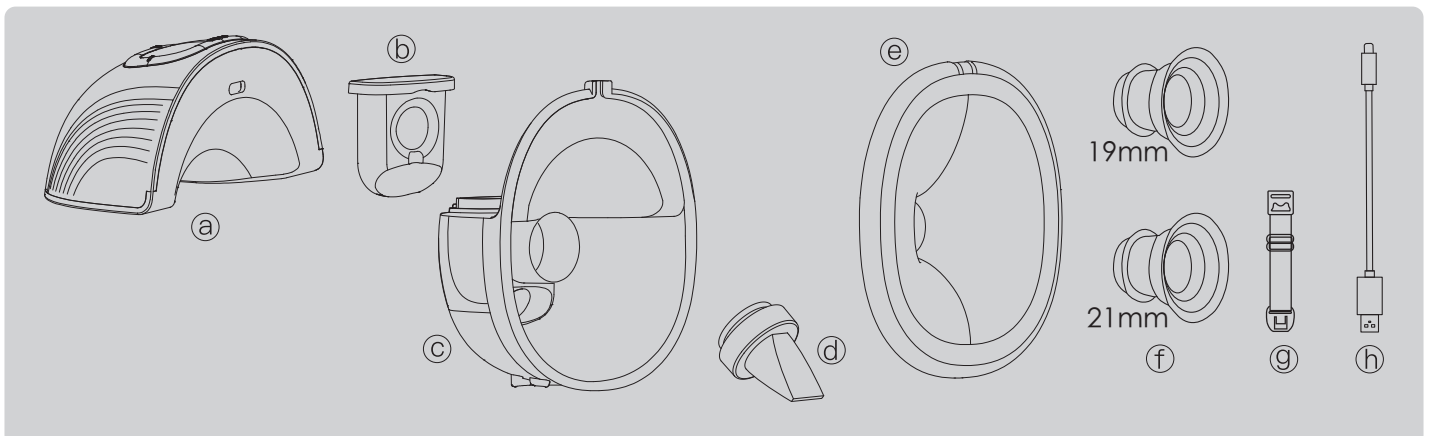


INSTALLATION

- Please sterilize all parts of the breast pump and wash your hands before installation.


1. Push duckbill valve onto milk collector, and make sure it is completely fastened.

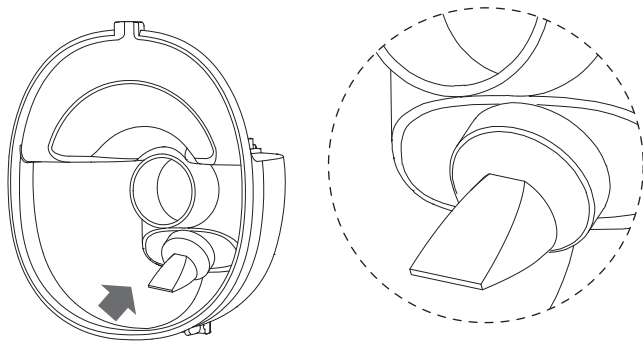


2. Put the silicone diaphragm into the milk collector, and press it with your hands to make the edges fasten tightly to ensure a perfect seal.

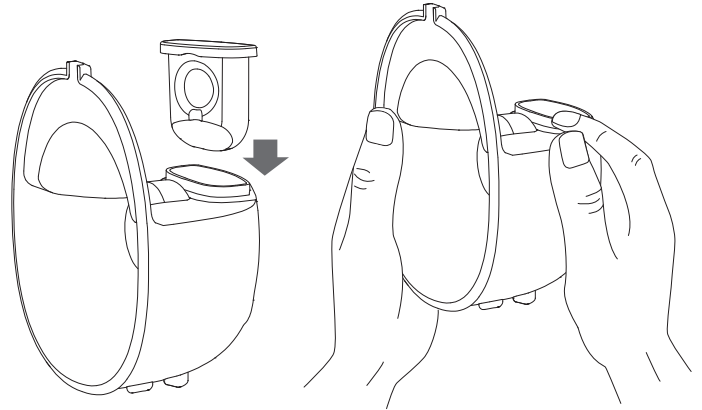


3. Install the flange on milk collector, make sure to align the diversion groove on the flange with the backflow groove on the milk collector, and then press the edges with both hands to ensure the seal is in place.

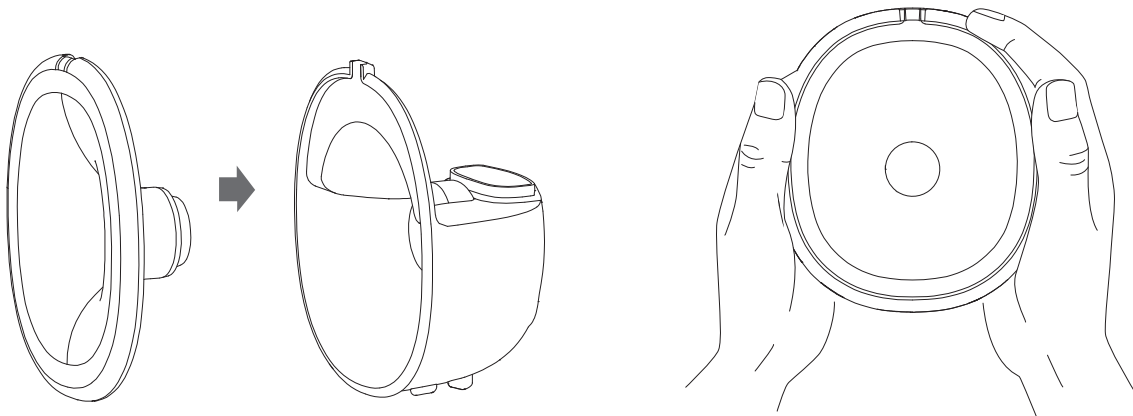

4. Install the motor part on the milk collector, align the notch under the motor with the silicone diaphragm, and assemble it downward to fit the milk collector to ensure a complete seal.

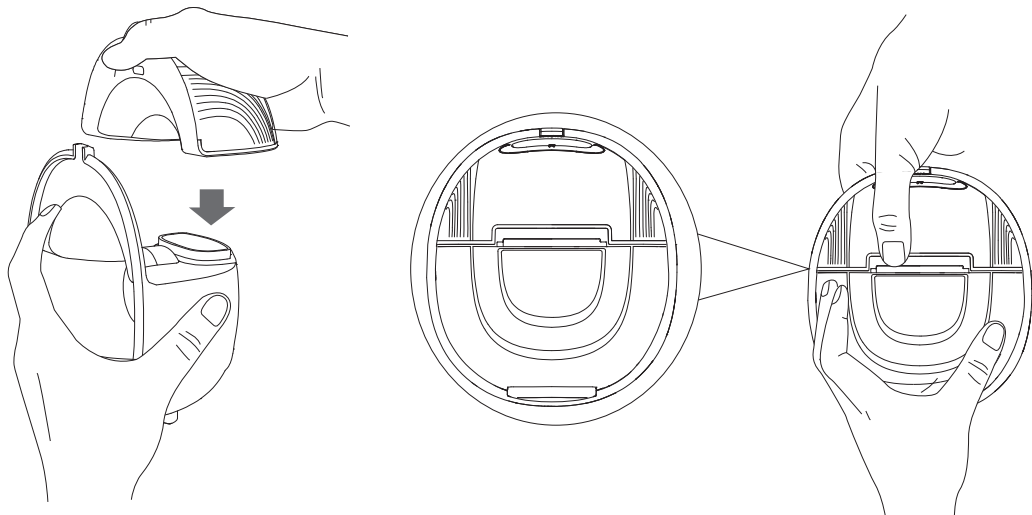


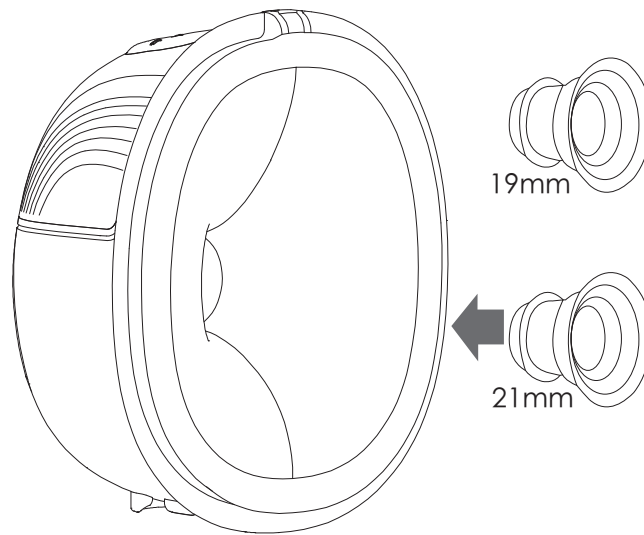

Measure your nipple size and choose the right flange size to suits you best.

19/21mm flange inserts are included in the box. Original pump body is 24mm. DO NOT exceed the maximum milk volume of collection cup when pumping.


Relax and sit in a comfortable chair, lean slightly forward (use a cushion for back support). Make sure to keep the breast pump and breast cup upright.

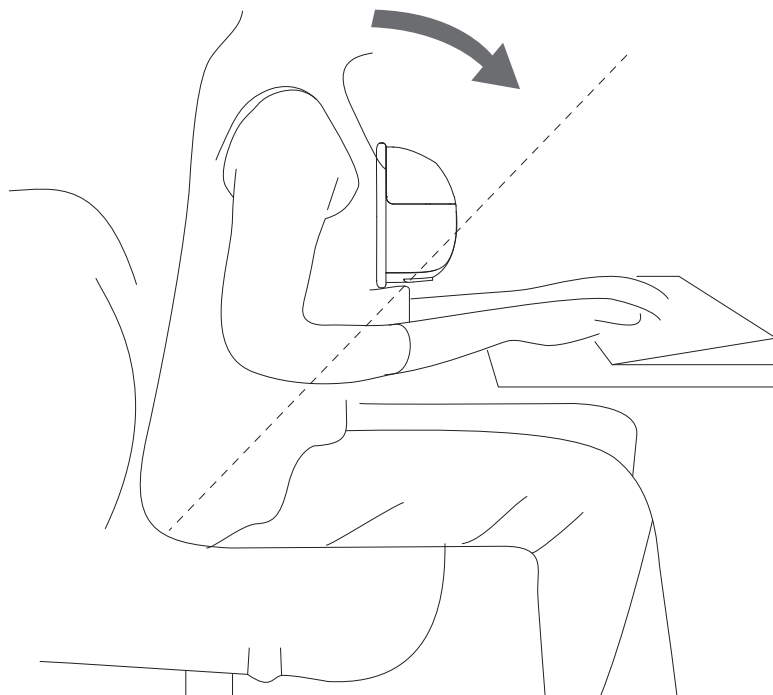

Sitting posture


INSTRUCTIONS FOR BRA ADJUSTING BUCKLE

Adjust the length of bra straps, make pumping more comfortable. Adjust the bra straps to improve comfort while pumping. Attach one end of the adjustment buckle to the inner buckle of the nursing bra, then secure the other end to the bra strap. Tight bra buckle on the other end of bra adjustment buckle.

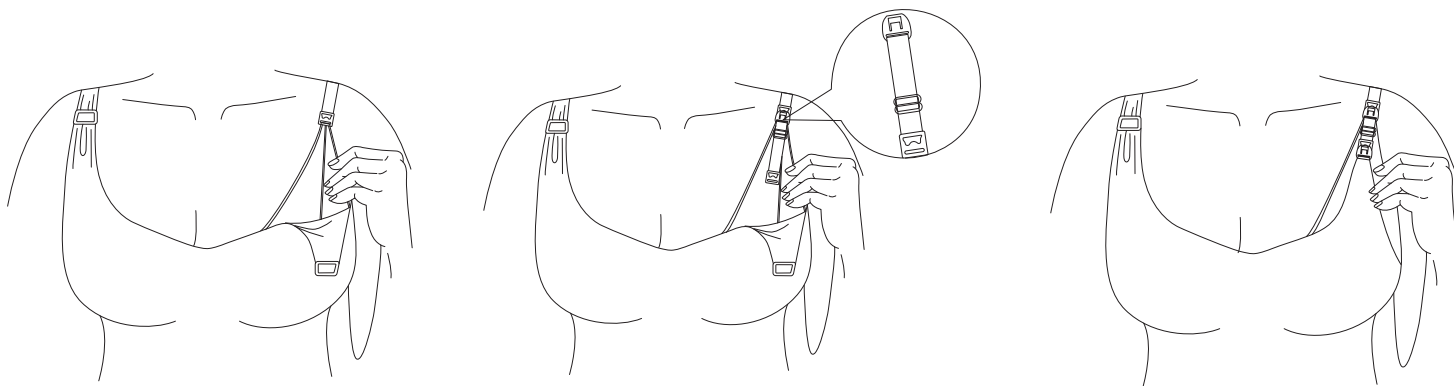


TO BEGIN PUMPING

Press the flange firmly against the breast, leave no gaps and make sure your nipples are centered. Keep it upright. Be careful not to distort the position of the product or your chest, as this may cause the breast pump to lose suction.

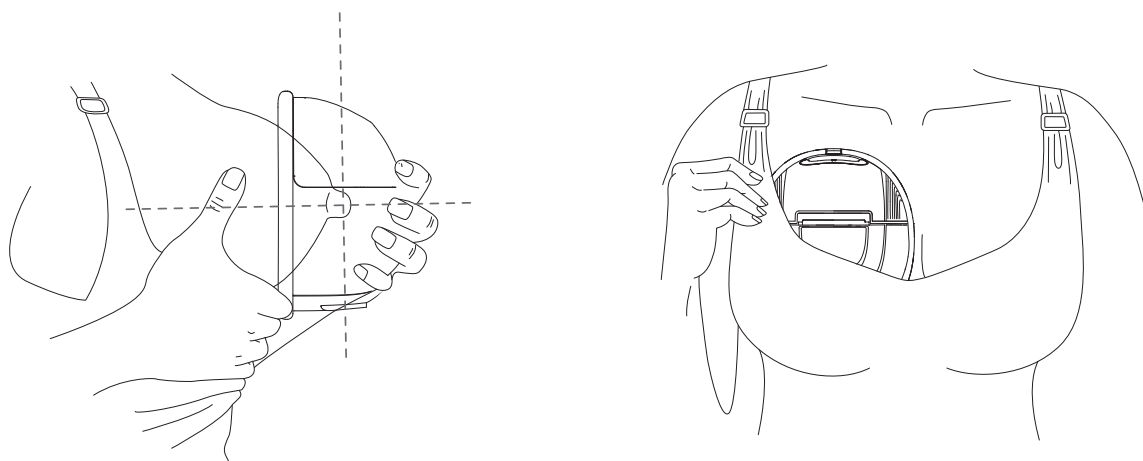

DISPLAY AND BUTTONS

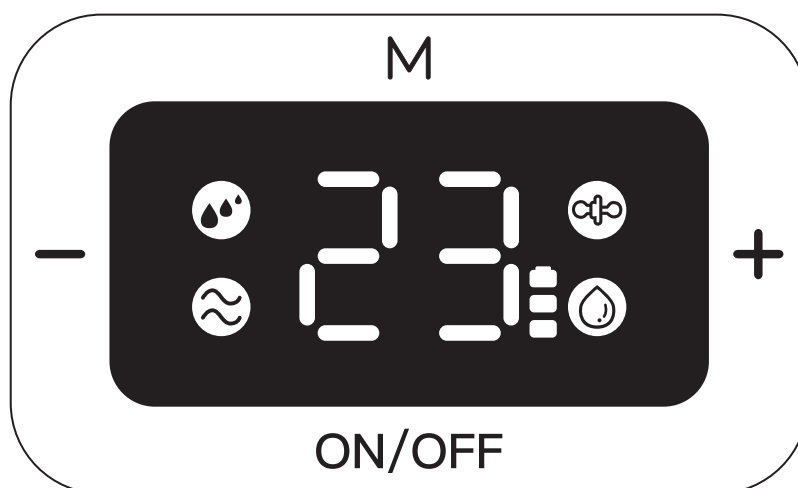

Button use instruction		
ON/OFF	ON/OFF Button	Long press to turn on/down. Short press to pause. The pump will automatically shut down after pause over 10 minutes.
M	Mode switch button	Default stimulation mode “M” when device first turned on. Added a built-in memory function, the pump will automatically enter the mode you used last time when device is turned on.
— +	Suction power switch button	All modes have 1 to 9 levels that can be chosen.

Display Instruction		
	Battery indicator	“  ” This symbol is flashing means the products should be charged as soon as possible. The device will auto shut down when the power is too low. The symbol will light up one by one while charging. Three symbols keep bright means the charge is finish and power is full.
	Time display	The device will auto shut down when it keep work for 30 minutes.
	Level display	1 to 9 levels.
	Stimulation Mode	Simulate NATURAL SUCKING of baby.
	Automatic Mode	Variable frequency to get MORE milk.
	Massage Mode	Gentle STIMULATION to get milk flow.
	Expression Mode	Slow and deep sucking to EMPTY OUT.

*** How to use:**

1. Ensure all parts of the breast pump were sterilized and installed correctly.
2. Please wash your hands and use a hot wet towel to massage your breast before use.
3. Press the flange firmly against the breast, leave no gaps and make sure your nipples are centered.
4. Keep it upright. Be careful not to distort the position of the product and the chest, otherwise the breast pump will have no suction.
5. Long press the power button to start. Choose the suction that makes you comfortable.

*** Tips:**

We suggest moms to try different modes and suction according to your actual breastfeeding situation.

Note:


This breast pump will automatically turn off after working 30 minutes.




ENDING YOUR PUMPING JOURNEY

After you finish pumping please remove the breast pump from the breast. Keep the breast pump upright and remove the motor part and hold the milk collector to pour the breast milk into the bottle. Make sure the groove of the collector cup is aligned with the mouth of the bottle to prevent breast milk from spilling.


1. When you have finished expressing, long press the power button to switch off and break the seal between the breast and the pump funnel with your finger, and then carefully remove the pump from your breast.

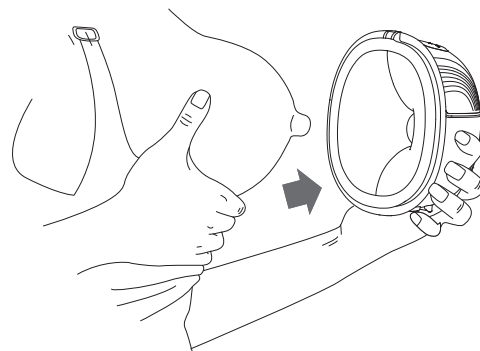

2. Separate the milk collector from the motor. Aligning the groove with the bottle, pour the milk into the bottle for feeding/storage.

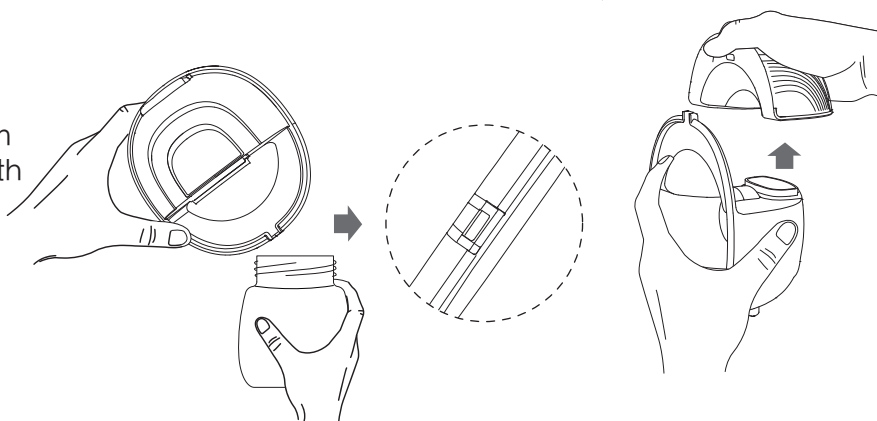

3. Clean other used parts except the electric motor and then disinfect.

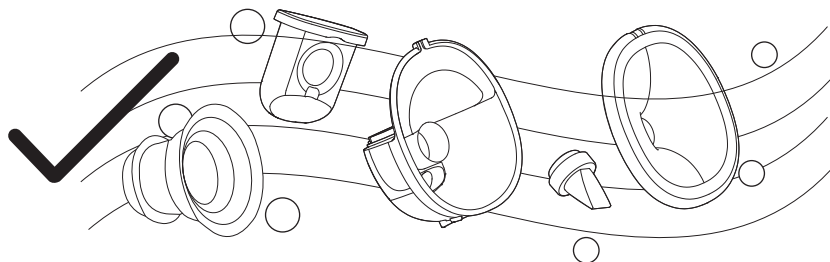

WORKING CONDITION

Rated temperature range: 5°C/41°F to 40°C/104°F

Rated relative humidity: 20%-80%

Rated atmospheric Pressure: 850Hpa - 1060Hpa

Long-term storage should be kept in a dry and cool place


TRANSPORT & STORAGE CONDITION

Temperature: -20°C/-4°F to 55°C/131°F

Humidity: 20% - 80%

Atmospheric pressure: 850Hpa - 1060Hpa

★ After cleaning and disinfection, the duckbill valve and suction diaphragm must be dried before use.

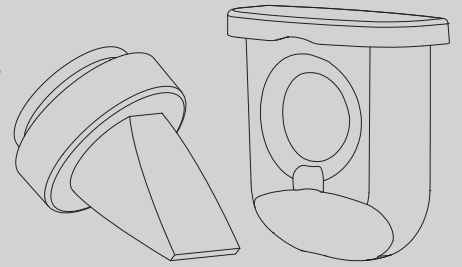

STORING BREAST MILK

Breast milk should be sealed and stored in refrigerator for long time storage.

Storage Ways	Storage Temperature	Longest Storage Time
Room	19℃/66.2°F~26℃/78.8°F	4~6 Hours
Refrigerator	<4℃/39.2°F	3~8 Days
Freezer	-15℃/5°F	14 Days
	-18℃/-0.4°F	3~6 Months
	-20℃/-4°F	3~6 Months

TO WARM MILK

1. In order to preserve the nutrition of the breast milk, you can warm the breast milk with 40℃/104°F warm water or use a bottle warmer.
2. Do not thaw frozen breast milk in the microwave oven or the boiling water (will damage the nutrition of breast milk).

SAFETY CLASSIFICATION

Properly dispose of the device in accordance with local, state and federal regulations.

- Type of protection against electric shock: Internally powered equipment.
- Degree of protection against electric shock: Type BF applied part.
- Safety degree of using in flammable anesthetic gas blending with air, oxygen or nitrous oxide: Non-AP/APG.
- Continuous or Non-continuous operation: Continuous.
- Classification of sterilization methods: No need for sterilization.

Symbol



Type BF
applied part



Stand by



Caution

PRODUCT SPECIFICATIONS

Name	Wearable Breast Pump	Model	AM8133	Battery Specification	3.7 V 500 mAh
Voltage	5V 	Place of Origin	China		
Power	5W	Warranty Time	1 year		

CAUTIONS

1. Please read the manual carefully before use.
2. Please fully charge the product before first use. Do not charge the battery pack for a long time. Lithium batteries can only be replaced by OMC or authorized service providers, and replacing the wrong model battery will have explosion dangers. Dispose the waste battery packs under local laws and regulations.
3. If the product is not used for a long time, the battery will run out of power. It needs to be plugged in for reactivation. wait for ten minutes for the light to indicate that is has been activated. Then continue charge the product till fully charged. If it is not activated for more than half an hour, please consult customer service.
4. The pump motor and other electric parts CAN NOT be washed in water or sterilized into steam sterilizer.
5. Please use the charging cable provided with this product. Do not plug the charging cable of any other manufacturer into the motor for use, so as not to cause damage to the motor.
6. Please use DC adapter, select the adapter with input voltage AC100-240V and output DC 5V and 1A.

CHARGING METHOD

When charging, please plug adapter into the power socket and make sure it is in place. When charging, motor should be turned OFF. Motor will not be charged when it is turned ON. During charging process, light will be flashing. When battery is fully charged, all lights will be on.



WARNINGS

The breast pump will not work when charging.

TROUBLE SHOOTING

Too little or no suction

- A. Check if all components are installed correctly.
- B. Check if the centre of flange aligns the nipple and clings to breasts, make sure no air entry.

WARRANTY CARD


Thank you for purchasing our product. Please keep your receipt as proof of purchase for any warranty claim.

For warranty claims and supports
please email us at lucinababy_amzus@hotmail.com

Name		Telephone	
Product Name		Model	
Purchasing Date			
Address			

NOTICE

- 1.This product comes with 1 YEAR warranty. Please contact the MANUFACTURER for warranty claims.
- 2.This warranty does not cover defects caused by: misuse, abuse, improper storage/alteration of the product, use of improper components, failure to comply with the operating instructions using the accessories supplied with the product, or normal wear and tear.
- 3.The user of this product assumes all responsibility and risk for the use of this product.
- 4.WARNING: No modification of this equipment is allowed.

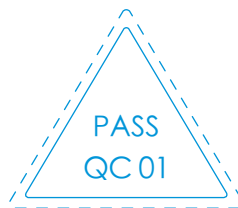

OMC Medical Ltd.

Add: Planet House, North Healh Lane Business Park, Horsham, RH12 5QE,
United Kingdom.

Website: www.oandkmuby.co.uk

Please read the manual carefully before use.

Please store the product where children can not reach.